

# Spray Off N



**Disinfection cloths for cleaning and disinfection of medical products and surfaces in all fields of medicine and company hygiene**

- ready for use, aldehyde free
- very rapidly and extensively effective in 30 seconds
- residue free, pleasant smell, good compatibility with all sorts of materials

Examined and certified according to VAH rules, bactericide, fungicide, virucide HBV/HIV/BVDV (HCV), Vaccinia, tuberculocide (incl. M.-terrae).

Distribution Austria:



**Meditrade Medicare Medizinprodukte GmbH & Co KG**  
 A-6330 Kufstein • Oberer Stadtplatz 6  
 Tel. +43(0)5372/61444 • Fax +43(0)5372/61444-4  
 info@meditrade.at • www.meditrade.at

Distribution Germany:



**SensoPharma Medizinprodukte & Pharmahandel GmbH**  
 D-83088 Kiefersfelden • Franz-Huber Str.52  
 Tel. +49(0)8033/303-094 • Fax +49(0)8033/303-195  
 info@sensopharma.de • www.sensopharma.de



# Spray Off N



## ■ Type of preparation:

Ready for use, aldehyde free. Very rapidly effective. For cleaning and disinfection of alcohol resistant surfaces of medical equipment and objects.

## ■ Range of Effects:

Bactericide, fungicide, virucide HBV/HIV/BVDV (HCV), Vaccinia, tuberculocide (including M.-terrae).

## ■ Fields of Application:

Ideal for all those fields where short residence times between treatment and intervention are required. In clinics, medical offices, ambulances, outpatients' departments and in the field of nursing, but also for laboratories and for use in the industrial, culinary and alimentary fields.

## ■ Application:

Wipe surfaces and objects carefully and leave the disinfectant to act for 30 seconds. Do not use for cleaning alcohol-sensitive surfaces and acrylic glass.

Examined and certified according to VAH rules. Expert opinion: Prof.med.R.Schubert, Frankfurt 2001, Prof.med.W.Solbach, Lübeck 2001. Prof.med.R.Schubert, Frankfurt 2003. HBV/HIV. Effectiveness: Dr.Steinmann, Bremen 2000 M.terrae-Wirksamkeit (M. terrae effectiveness) Hygiene Nord<sup>1</sup> Greifswald 2004.

<sup>1</sup> Hygiene Nord Inc. = Independent microbiological test laboratory in the field of hygiene in Germany

## ■ Composition:

Saturated with solution, 100ml contain:

10,0g of ethanol / 30,0g of 1-propanol / 15,0g 2-propanol

0,05g of didecyl dimethyl ammonium propional / 0,02g of alkylamine guanidine diacetate.

Attention: Be careful when using on alcohol-sensitive materials. Inflammable. Keep away from sources of ignition. Irritant, if it gets into your eyes rinse them thoroughly with water and consult a doctor. Use in properly ventilated areas only. Keep away from children.

Toxicology: Acute toxicity: LD 50 => 8000 mg/kg. Rat oral.

Local tolerance: Good cutaneous tolerance in case of short time contact.

Chemical and physical data: pH value: approximately 6,5-7/density: 20°C:-0,880-0,900g/cm<sup>3</sup>.

Article number	Shipping		Box
30067	150 pieces	(Can)	12 x 150 pieces
30066	150 pieces	(Refilling)	12 x 150 pieces



Distribution Austria:



**Meditrade Medicare Medizinprodukte GmbH & Co KG**  
A-6330 Kufstein • Oberer Stadtplatz 6  
Tel. +43(0)5372/61444 • Fax +43(0)5372/61444-4  
info@meditrade.at • www.meditrade.at

Distribution Germany:



**SensoPharma Medizinprodukte & Pharmahandel GmbH**  
D-83088 Kiefersfelden • Franz-Huber Str.52  
Tel. +49(0)8033/303-094 • Fax +49(0)8033/303-195  
info@sensopharma.de • www.sensopharma.de